




TT DTIRA

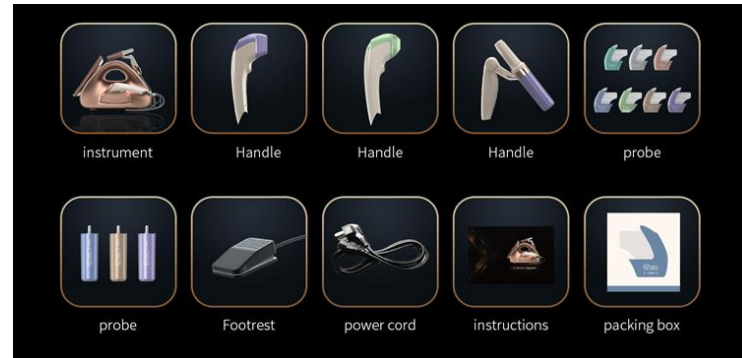

MICRO-PULSE
OPERATIONTECHN
OLOGY&PROBE

CIRCULAR
OPERATIONOPTIMIZAT
ION HAND
TOOL&PROBE

TT DTIRA 13.3 inch

HD smart screen

Full dynamic design | human-computer interaction
Removable plug | Full sense of technology
sense of technologyPatented


MODEL	KSH204
Power Supply	AC 110VAC 220V
Power :	650W
Frequency :	2MHz4MHz5.5MHz
Instrument weight :	10kg
Package size :	60 X 43 X 57 cm
Gross pro duct quantity	20kg



MODEL	KSH209
output frequency	1MHz-7MHz
Output voltage	10-200W
Input voltage	AC 110V-220V
Fuse	5A
Size of Air box	64×47×67m
Weight	About39kg

SEVEN-IN-ONE

(Frozen HIFU+ Vmax+4D multi-row
+Privacy+Detection
function+Liposonic+ Golden RF)

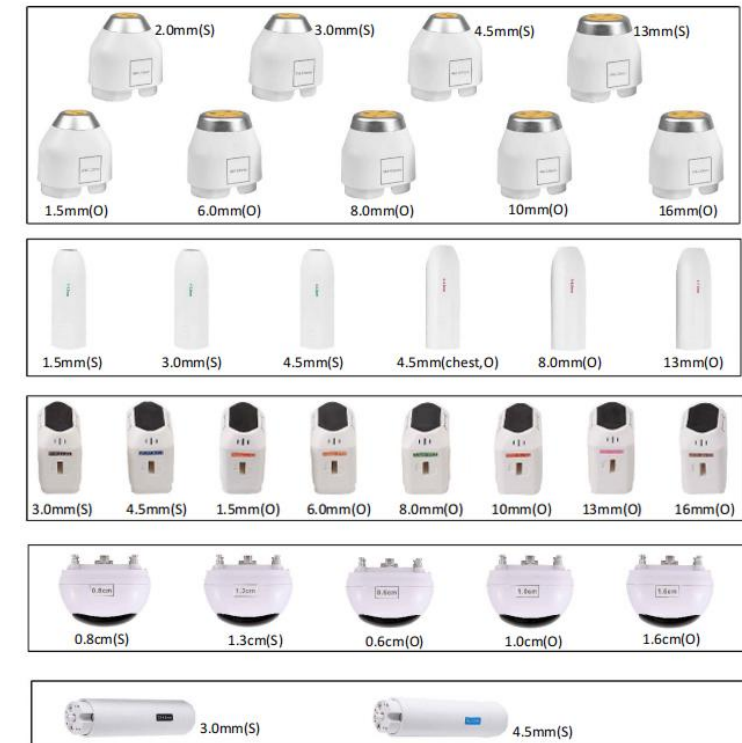

Type of probe	Function
	10p: It is suitable for dead corner and a striated stretch mark.
	25p: Apply to face.
	64p: Apply to the body.
	Nano: Import,lifting ,tightening.

7D Focused Ultrasound Manual

What is 7D Focused Ultrasound?

MMFU(Macro & Micro Focused Ultrasound) : "Macro & Micro High Intensity Focused Ultrasound System" Non-Surgical Treatment Of Face Lifting, Body Firming And Body Contouring System!



Cartridges*7



Handle*2

AME90



Power Supply




Accessories rack*3



AME72

HIFU SYSTEM

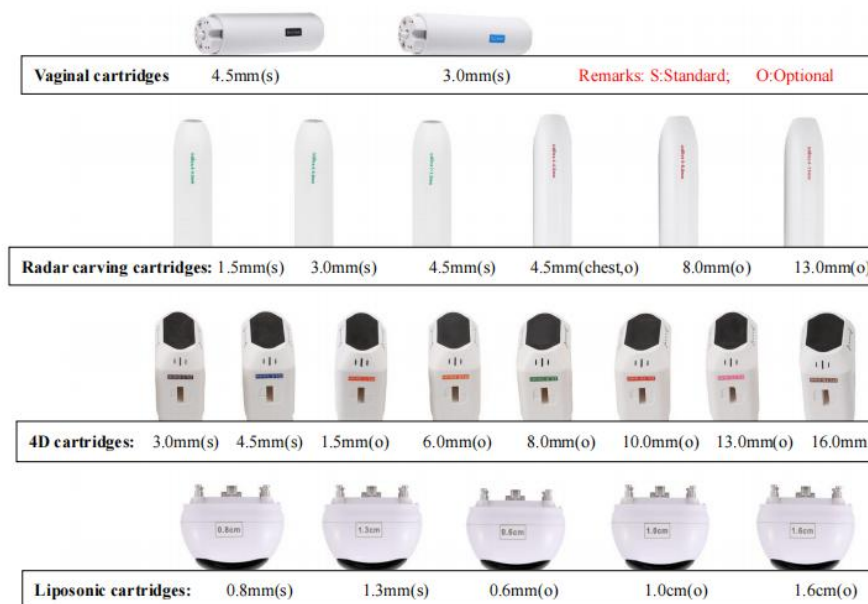

SIX-IN-ONE

(Radar carving+4D multi-row+vaginal tightness+detection+liposonic+RF)


MODEL	AME72
Product name	Six in one
output frequency	1MHz-7MHz
Output voltage	10-200W
Input voltage	AC 110V-220V 50Hz/60Hz
Package size :	60 X 43 X 57 cm
Fuse	5A



Facial anti-aging



F Cartridge

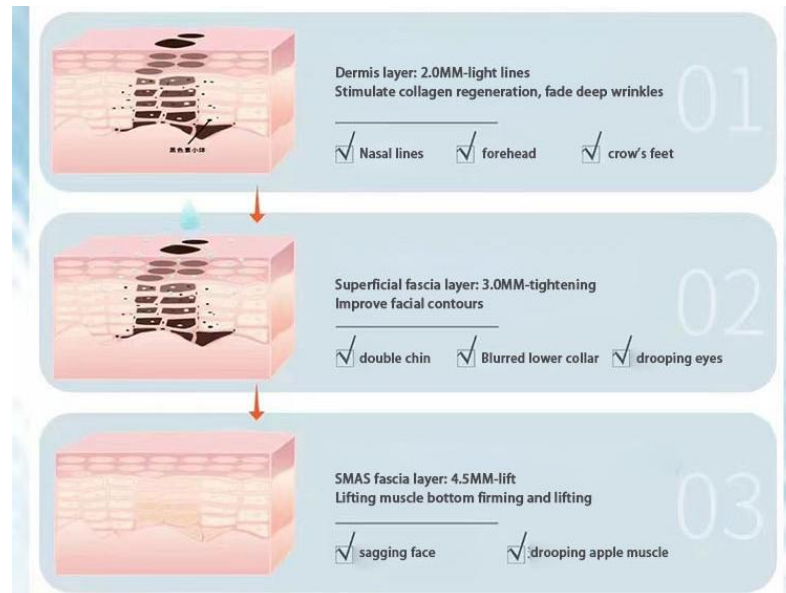

Function:

- 1, Suspended lifting tightens and instantly activates the collagen and tissues of the muscle bottom;
- 2, Instantly enhance facial contours, treat body parts to tighten, adjust and shape curves;
- 3, Improve tiger lines, forehead lines, lift eyebrows and apple muscles, tighten double chin, fine pores, and even white skin;
- 4, Exclusive 2mm professional eye probe for the treatment of dark circles under the eyes, crow's feet and other fine lines;



MODEL	AME67
Product name	HIFU 7D
Fluence	0.1~3.0J(0.1step)
Cartridge	7Mhz.4Mhz (1.5mm, 3mm, 4.5mm)
Spacing	1.0 -2.0mm(0.5mm/step)
Length	5-25mm(5mm/step)
Dimensions	500x515x1310(WxDxH)

MODEL	AME67
Product name	HIFU 7D
Fluence	0.1~3.0J(0.1step)
Cartridge	7Mhz.4Mhz (1.5mm, 3mm, 4.5mm)
Spacing	1.0 -2.0mm(0.5mm/step)
Length	5-25mm(5mm/step)
Dimensions	500x515x1310(WxDxH)







SHOCKWAVE

Classic application



Knee



Elbow



Shoulder



Hip



Pubic Symphysis



Hand/Wrist



Lumber Region




Foot/Ankle



Fat Accumulate

SHOCKWAVE MACHINE

Spreads In The Tissue

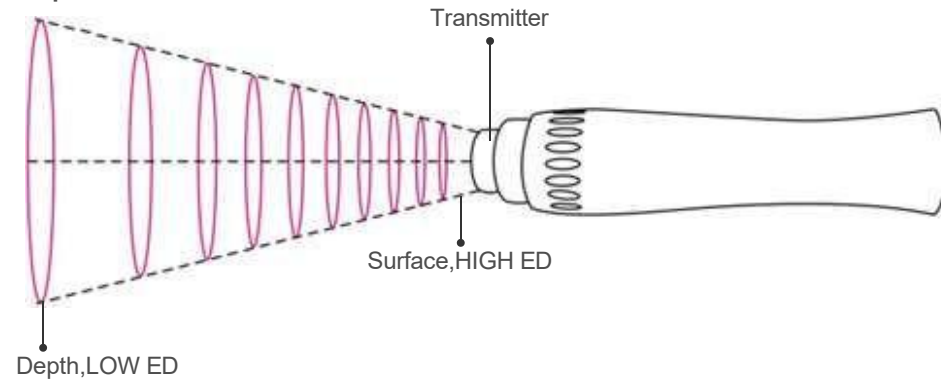

Five features to give you peace of mind
Multi-functionality Easy Experience

- ▶ High performance compression systems
- ▶ No noticeable noise
- ▶ One machine for two machines
- ▶ Certifications
- ▶ Extra large penetration power

Say goodbye to soreness, pain and tiredness and relieve various symptoms of the body





Classic Indications

1. Shoulder pain, Calcific tendonitis;
2. Radial or ulnar humeral epicondylitis;
3. Trochanteric tendonitis;
4. Waist or stern pain;
5. Patellar tendonitis;
6. Tibial stress syndrome;
7. Achillodynia;
8. Plantar fasciitis;
9. Broken fat.

Advantages


1. High effectiveness, fast relieve pain.
2. Noninvasive, no anesthetic, convenient and fast.
3. Each nursing only 20 minutes, 5-10 nursing each session.
4. Widely applied to different soft tissue treatments.






Suit for different body treated area

(underarm, forehead, chin, beard, chest, back, legs, arms, bikini)

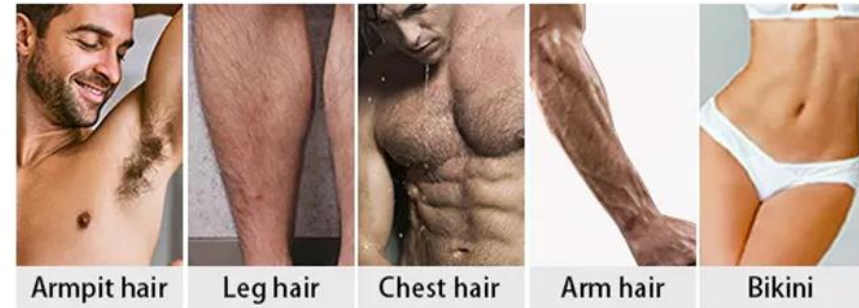


808 DIODE LASER HAIR REMOVAL


During a laser hair removal treatment, light passes through the skin and is absorbed by the melanin in the hair shaft for regrowth, hair follicle and thermally destroys the cells responsible. After up to 3 treatment course,you can get rid of the unwanted hair permanently.

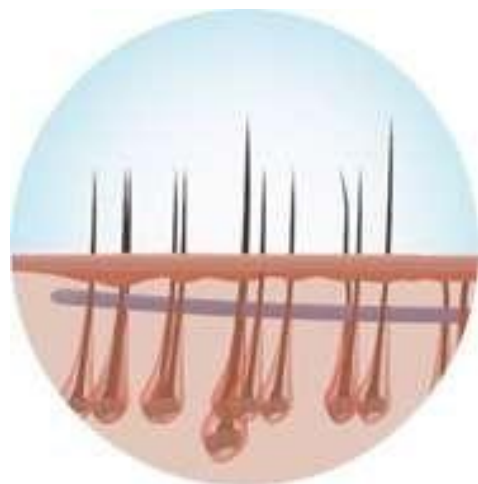

Before

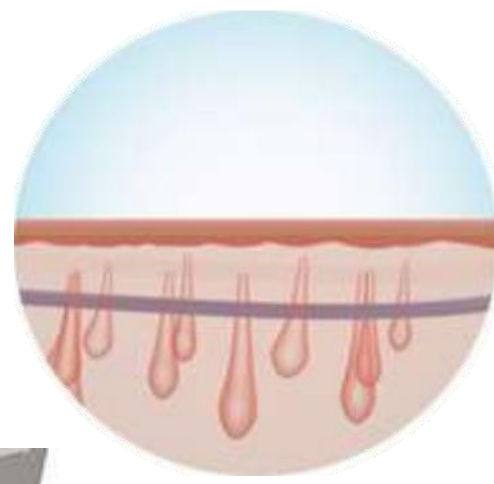

After

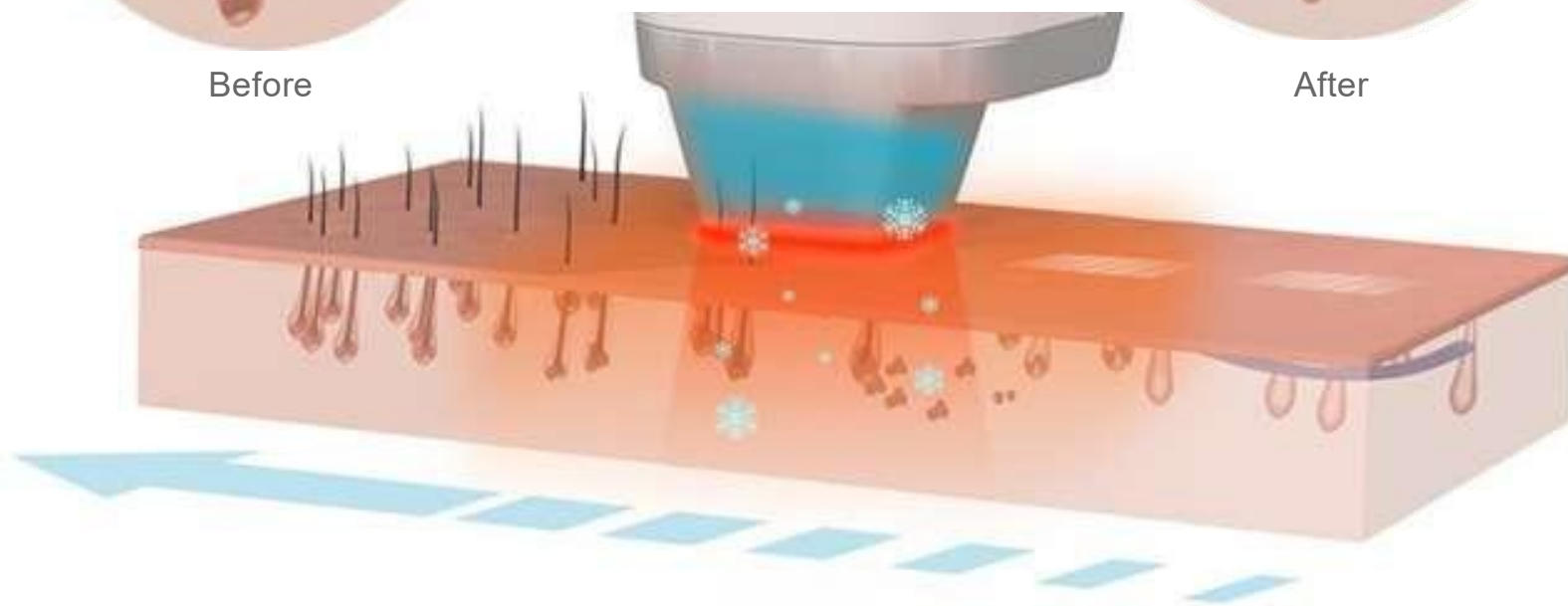

808nm Laser Advantage

1. Germany Imported Laser Bars
2. Symmetrical Rectangle Spot
3. High Repetition Frequency
4. Nitrogen-sealed
5. Advanced Cooling
6. Long Life Span
7. Continuous Working
8. Program Controlled Pulse Width

808nm Diode Laser Handpiece

Hair follicle melanin by specific wavelength of laser absorption, and the heat exchange to destroy hair follicle,the laser energy can be shot down into the hair follicles within the hair follicle is destroyed energy decomposition, so that the Hair follicles to stop growing, because the laser energy about 15% of the skin can penetrate the dermis layer, the epidermis and more with melano-ma of the hair follicle deep through the heat damage to hair follicle degeneration and atrophy; achieve permanent hair removal results,excrete by the process of metabolism, and the vessel and nerve will not be damaged.

MODEL	KSH87
Energy Density	0~120J/cm2
Pulse Width	0~600ms
Repetition Rate	1~10Hz
Skin Cooling Temperature	-5C--5C by cooled Sapphire
LCD Screen	15
Input VAC	220VAC/50-60Hz or 110VAC/50-60Hz




Function:

Hair removal & Skin rejuvenation: Our machine combines the benefits of all 3 wavelengths 808nm 755nm 1064nm into one handle, 3 wavelengths have different laser characteristics. Really reach: painless, fast, effective and suitable for all kinds of skin type easily.

Applications:

1. Treat all kind of hair color from black hair to white hair
2. Treat all skin types
3. No pain and shorter treatment sessions
4. Effective and safe treatment for permanent hair removal

Voltage		110/220V, 50-60Hz	
Power		3000W	
Screen		15inch touch screen	
Cooling System		Water cooling, air cooling, semiconductor	
RF	Frequency	1MHz	
	Power	5-50J/cm ²	
E-LIGHT	Wavelength	480-1200nm, 530-1200nm, 640-1200nm	
	Spot Size	12mm*30mm	
	Power	1-50J/cm ²	
Pico Laser	Frequency	1-5Hz	
	Wavelength	1320nm, 1064nm, 532nm	
	Spot Site	5mm-10mm	
	Energy	20-1000mJ	
		Frequency	1-10Hz
Machine Size		76.5cm*57.5cm*136.5cm	
Package Size		92cm*70cm*115cm	

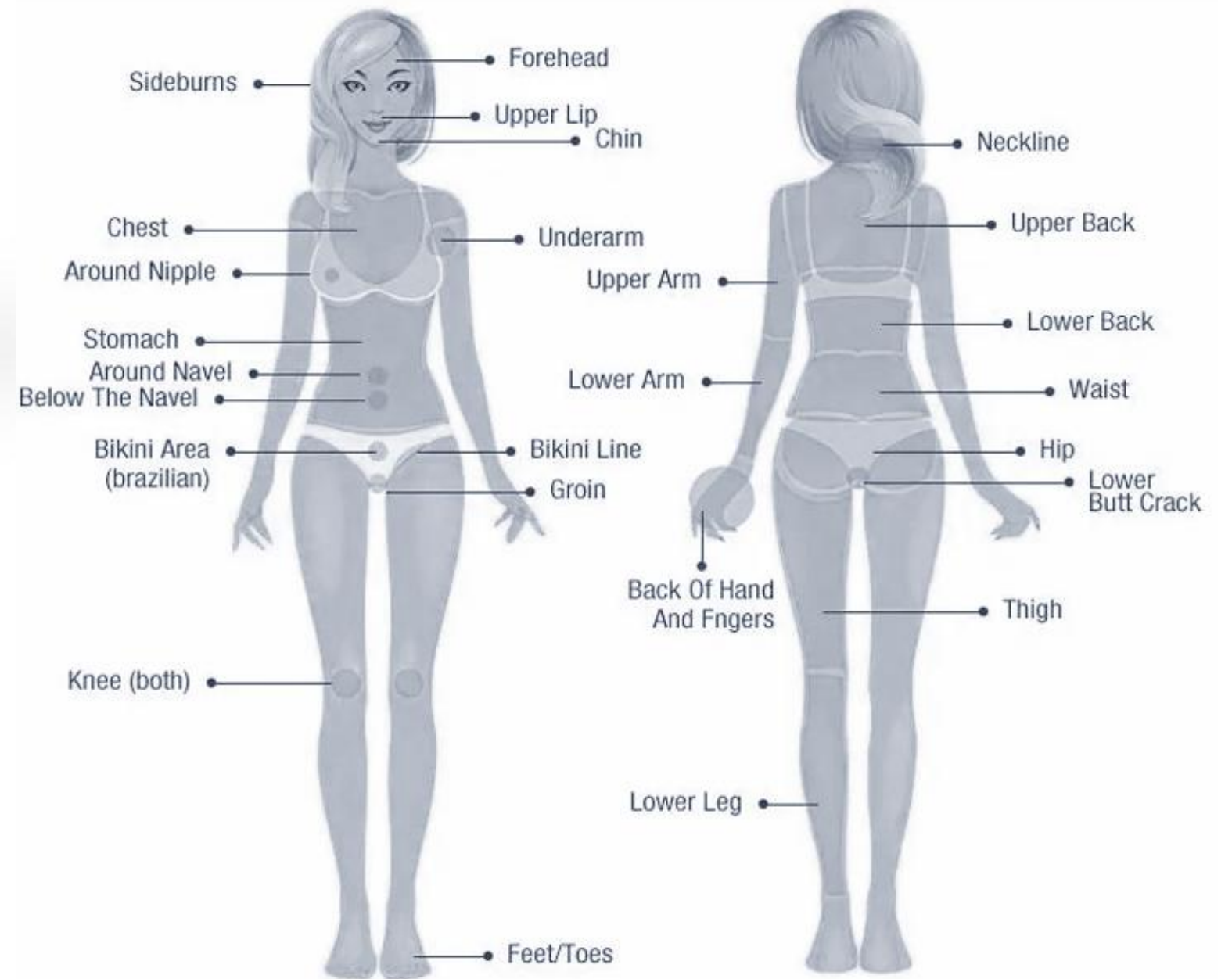



Hair Removal handle

Using sapphire crystal, semiconductor target laser, hair removal effect is better, Rejuvenation improves skin elasticity, cools freezing point hair removal.



CRYOLIPOLYSIS SYSTEMTM



TRUE CRYOLIPOLYSIS TECHNOLOGY

PRINCIPLE

As triglyceride in fat be converted into solid in particular low temperature, it uses advanced cooling technology to target fat bulges selectively and eliminate fat cells through a gradual process that does not harm the surrounding tissues, reduce unwanted fat. When fat cell are exposed to precise cooling, they trigger a process of nature removal that gradually reduces the thickness of the fat layer, and the fat cells in the treated area are gently eliminated through the body's normal metabolism process, to eliminate unwanted fat.

FREEZE FAT RENDERINGS



45 min treatment



1 day after treatment



2 day after the results



15 days after the results



Advantages

1. The fourth generation of frozen weight loss techniques
2. Three work modes
3. The parental thermal separation technology
4. Anti-frostbite simultaneous working technology
5. Dual-head simultaneous working technology
6. Automatic fault detection technology
7. Intelligent vacuum stabilization technology
8. The exclusive technology foe vendors







picolaser tattoo removal machine

Application:

- 1.All color tattoo, eyeliner and lip liner remove
- 2.Epidermis and dermis pigment remove
- 3.Nevus of Ota. Blue nevus black nevus, coffee spot removal
- 4.Senile plaques, freckle, sunburn spot and chloasma remover
- 5.Skin whitening and wrinkle remove etc



Laser type	Emerald laser
Laser wavelength	1064nm, 532nm, 755nm, non-invasive
Highest energy	0.7J/cm ² , 0.7J/cm ² , 1.0J/cm ²
Frequency	1-10HZ
Voltage power	AC220V, 50Hz
Pulse width	600ps
Power	2000W
Spot diameter	2-10mm



Power	2000W
Model of overcurrent protector	220V, 10A
Working laser wavelength	1064nm; 755nm; 532nm; 1320nm
Laser output frequency	1-10hz
Spot diameter	2-10mm adjustable
Maximum output energy of single pulse	1064nm (100-1200mj); 755nm (100-1200mj); 532nm (50-600mj)
Eye hazard distance	NOHD is 100m, it is harmful to eyes in a sealed operating room, and protective glasses must be worn!



Wavelength:	10600nm
Laser apparatus	Sealed off laser device simulated by direct current
Output power	40w
Spot size	0.02~0.05mm ²
Aiming beam	Red semiconductor laser (635nm, less than 5mw)
Beam transport device	Articulated arm with 7-joint and balance weight
Output Mode	Continuous, single pulse, interval pulse and super pulse
Scan scope	Max 20mmx20mm
Scan graphics	Circle, triangle, square, rectangle, hexagon, ellipse, line
Scanning Mode	Random, Normal and Midsplit scan
Scanning speed	More than 10m/s
Distance	0.1-2.6mm
Cooling system	air circulation





Wavelength	1320nm & 1064nm & 532nm
Energy	1-2000mJ
Laser Type	Q-switched Nd:YAG
Spot Size	2-8mm
Repetition Rate	1-10Hz
Pulse Width	<10ns
YAG Bar Size	6
Power	500W
Aiming Beam	5mW red diode laser, 635 nm
Stand-by Working	Continuously for 12 hours
Scanning speed	More than 10m/s
Cooling	Self-contained, closed water cycle
Display	4" Dual Color LCD Touch Screen




1064nm: effective in treating endogenous pigmented skin lesions. Wash eyebrows, tattoos(dark and blue) and dark pigments, remove moles.
 532nm: effective for tattoos(red and brown), reddish pigments and age spots.
 755nm(honeycomb laser head): High absorption value of melanin, used for epidermis, sunburn, subcutaneous pigment, chloasma, tattoo, etc. Skin rejuvenation, stimulate the proliferation of collagen, eliminate acne, fine lines, shrink pores, etc.
 1320nm(black doll): skin rejuvenation, anti-aging, skin tightening, pore shrinking and deep cleaning.

Working Theory

Using the light blasting effect, the high-intensity laser penetrate the epidermis and can reach the pigment clusters in the dermis layer. Because the energy has a short time of action and the energy is extremely high, the pigment clusters will quickly expand and explode after absorbing the high energy in an instant. After the particles are swallowed by macrophages, excreted, and the pigment gradually fades and disappears.

The picosecond laser with ultra-short pulse width can effectively produce photo-mechanical effects and break the pigment particles into small fragments. Compared with nano-scale Q-switched laser, picosecond laser only needs lower energy to achieve the effect. It takes less number of treatment courses to achieve better treatment effect. Stubborn green and blue tattoos can also be effectively removed. Treated but incomplete tattoo removal, picosecond laser can also treat. In the mechanism of pigment particle destruction, there are mainly photothermal and photomechanical effects. The shorter the pulse width, the weaker the effect of converting light into heat. Instead, the photomechanical effect is used, so picoseconds can effectively crush the pigment particles, Resulting in better pigment removal.

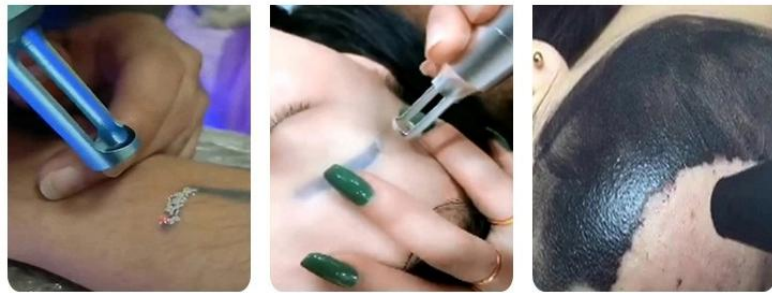

Tattoo Removal Eyebrow Removal Dark Face Doll



SUPER PICOSECOND LASER BEAUTY INSTRUMENT

- Remove tattoo or all colors
- eyeliner & lip line
- Remove pigmented
- Remove tattoo an eyebrow
- Skin beautifying and freckle removal



Co2 laser & acne scar removal machine

- Skin Tightening
- Face Lift
- Acne Treatment



Before and after



NEW
picosecond laser
Source spot removal/
fast/strong ability
 Vascular Removal | pigmentation removal

Tattoo Removal Eyebrow Removal Dark Face Doll

SUPER PICOSECOND LASER BEAUTY INSTRUMENT

- ◆ Remove tattoo or all colors
- ◆ eyeliner & lip line
- ◆ Remove pigmented
- ◆ Remove tattoo an eyebrow
- ◆ Skin beautifying and freckle removal





Multi-functional Hydradermabrasion Facial Beauty Equipment

INTELLIGENT OPERATING SYSTEM
EASY TO OPERATE




LARGE SCREEN
15 inch Touch screen




SOME DIFFERENCES COMPARE WITH OTHERS

Inside of the machine

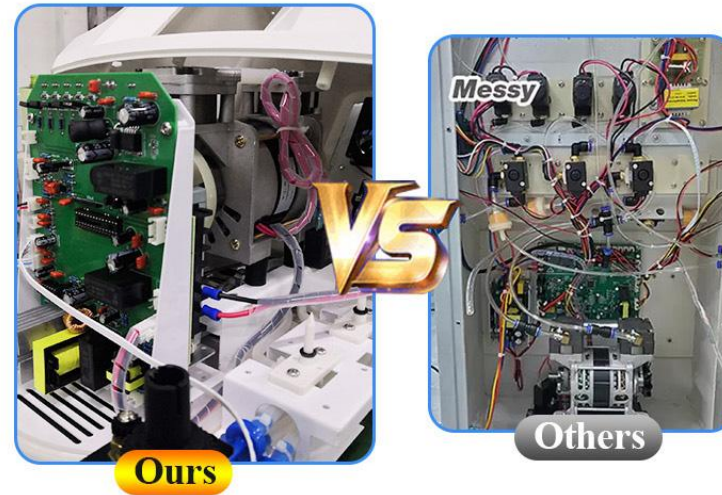

Air pump

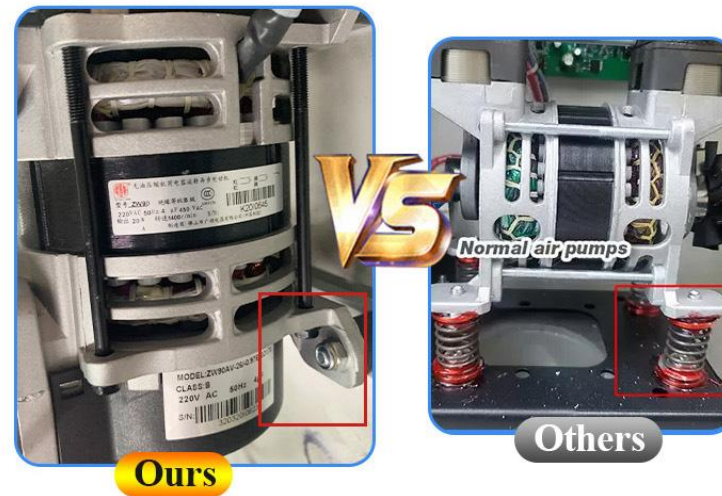



HIGH FREQUENCY


DiffGenerates high frequency current through the glass electrode tube, promotes blood circulation and promotes metabolism, Acne removal, skin care, hair care erent



LED LIGHT


It's appropriate for normal skin to acne skin & atopic dermatitis using high performance LED light.

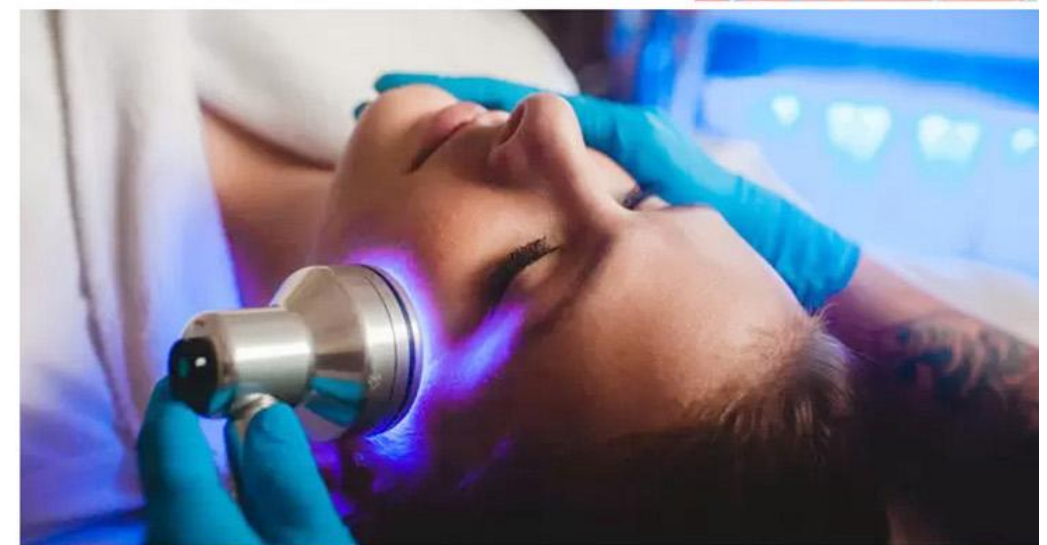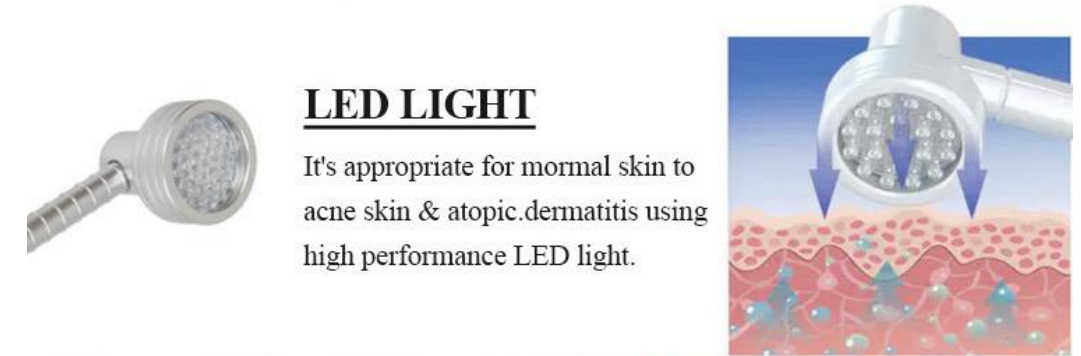

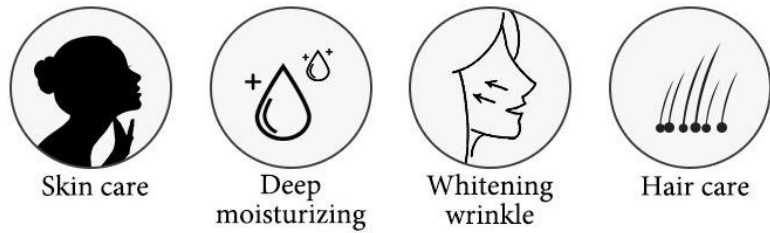

BODY COMPOSITION

Human body element analyzer

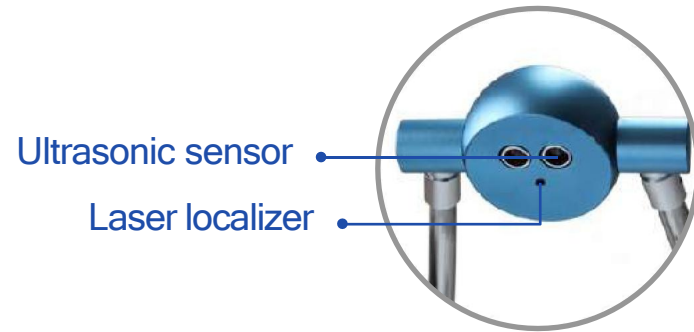

Hand electrode test Foot electrode test A4 color printer

Human body elements analysis equipment, apply the accurate measurement of AVR micro computer controller, bases on new statistics method DXA, analysis human elements: fat, weight, BMI (body mass index), non-fat and other health indicators through multi-frequency bioelectrical impedance analysis method (MFBI), give instruction on guests' health, and provide a scientific basis for losing weight effectively, which is considered as an epoch-making results of the health industry. It is health consultants, and it provide an independent analysis of health data for each test. Applicable to major fitness, body centers, medical centers, senior clubs, supporting projects of losing weight.



BODY BUILDING WEIGHT TEST SYSTEM



Data function

- Grasp the slimming effect to make the slimming treatment, providing the scientific basis.
- The reasonable ease and weight control help maintain the best physical strength Distinguish between muscle type overweight and obesity, find the best control weight method.
- Direct energy intake and dietary intake of nutrients in proportion.
- Guiding take nutrition recovery measures to promote the hard synthesis muscles.
- Evaluation the rehabilitation treatment effect, guiding scientific body slimming with scientific training, rehabilitation, and nutrition.

AC 110V-220V 50Hz-60Hz	
INPUT	
POWER	53VA
RATED CURRENT	<180μA
PRINTER	Micro+color
G.W.	110KG
SIZE	63*54*163cm

Data function

- Understand and detection teenagers the status of the physical and health, developing their the feasible exercise program.
- Understand the environment, nutrition and other factors on body composition effects.
- Know about the difference of similar others group ingredients.
- As reduce cardiovascular, diabetes and some cancers and other chronic disease incidence of the important preventive measures.

AC 110V-220V 50Hz-60Hz	
INPUT	
POWER	53VA
RATED CURRENT	<180μA
PRINTER	Micro+color
G.W.	48KG
SIZE	48*50*121cm



skin analyzer

- 36 MILLION IMPORTED CAMERAS
- BLUE LIGHT STERILIZATION
- EIGHT SPECTRAL IMAGING
- SKIN DIGITAL ANALYSIS

NEW

High Technology Skin Test Analyzer

- 8 Spectral Imaging
- 3D Simulation Slice
- Smart Solution Recommendation
- Skin Aging Prediction
- Cloud Sync Storage

moreme

Skin Analyzer

Scalp Testers / Skin Testers
Can meet your multiple service needs



Scalp Care equipment

BEAUTYWISE NEW MONEY-SEEKING PROJECTS

- Scalp detection
- nano spray liquid
- high-frequency comb
- massage care
- meridian brush head

USB direct connection
hair follicle/scalp/skin detector


USB direct connection for cell phone (Android) / tablet (Android) / PC

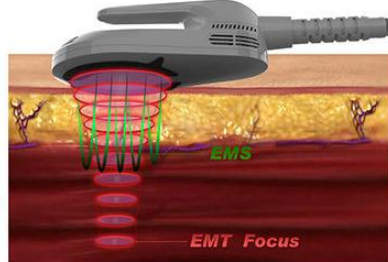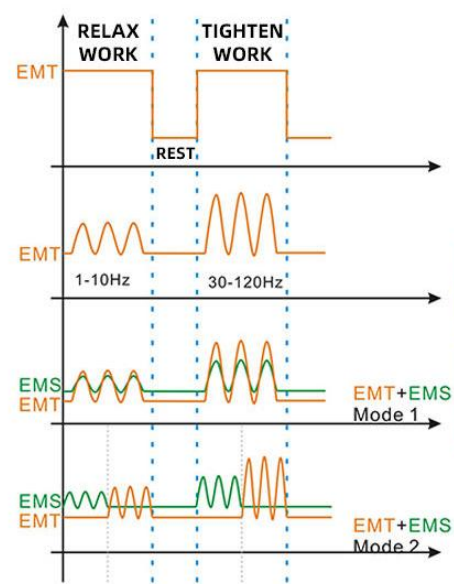

EMS+EMT



NEW SLIMMING MACHINE

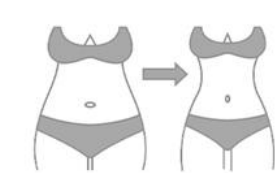

HI-EMT:
 designed for aesthetic purpose, having 4 applicators with higher intensity. It's cutting-edge technology in non-invasive body contouring, as it not only BURNS FAT, but also BUILDS MUSCLE.

EMS:
 Heat uniformly for the treatment area, so that the subcutaneous fat quickly reaches the treatment temperature, achieving the dual effects of firming and dissolving fat.





SHAPING THE PERFECT BODY GOOD SHAPE YOU CAN ALSO HAVE

study on Designing Method of
Double-elliptical Trainers




Thin arm
after use



Thin arm
after use



Thin arm
after use

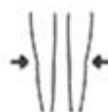

Thin arm
after use






TPH 441 10D laser

Two Shelf Can Choose

Medical Shelf

Automatic lifting

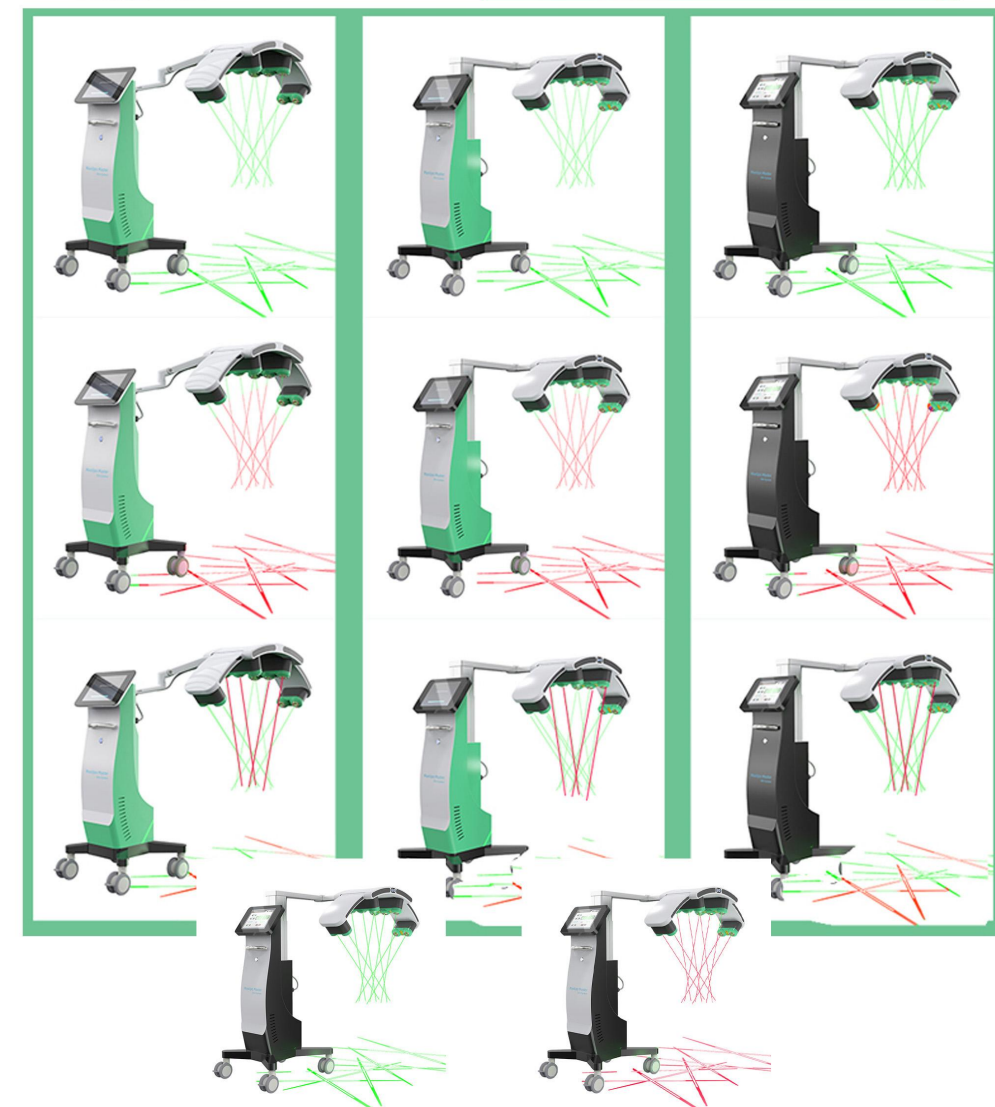

6D laser(KSH94)



635nm RED

532nm Green

635nm RED

532nm Green

6D laser+CRYO(KSH115)



635nm RED

532nm Green

635nm RED

532nm Green

6D laser+EMS(KSH156)



635nm RED

532nm Green

635nm RED

532nm Green

KSH216 10D+CRYO

10D Maxlipo Master Laser And EMS Cryo plate Weight Loss Machine






8D laser (AMP232)

8D Laser Shape Slimming Equipment

- ▶ Eliminate excess fat from abdomen, waist, back, buttocks, thighs, arms, double chin, etc.
- ▶ No anesthesia, no pain, no numbness, no scars, no trauma, no recovery.
- ▶ Four green lamps + Four red lamps

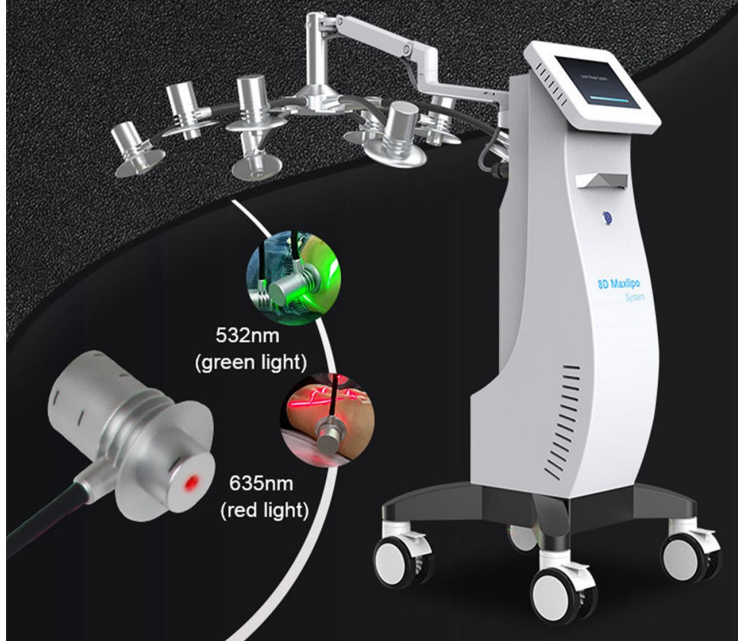

With the 532nm and 635nm cold source laser, the 8 adjustable laser heads target the fat cell beneath the treatment areas, creating a temporary pore within the fat cell, the fatty liquids can drain out of the cell, the volume of the fat cell is reduced. The fatty acids and glycerol will be flushed out naturally through lymphatic metabolism.



Vest line



Thin back



Thin Arm



Peach Hip

AMP269



KSY104





EMRF
EMRF FACE







Vacuum RF laser cavitation massage body slimming machine



The Vacuum series is a vacuum negative pressure , bipolar RF , infrared laser , skin folding , mechanical rolling technology instrument , suitable for all parts of the body of fat-dissolving,shaping the body, firming the skin, and physiotherapy equipment . It can effectively degrade the subcutaneous fat cells into carbon dioxide and water,and at the same time promote the metabolism of the human body, greatly improving the effect of lipolysis, shaping and physiotherapy.

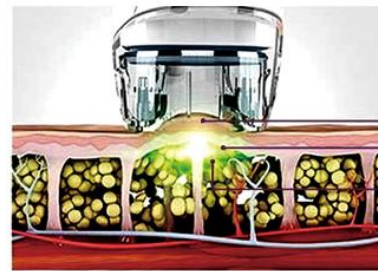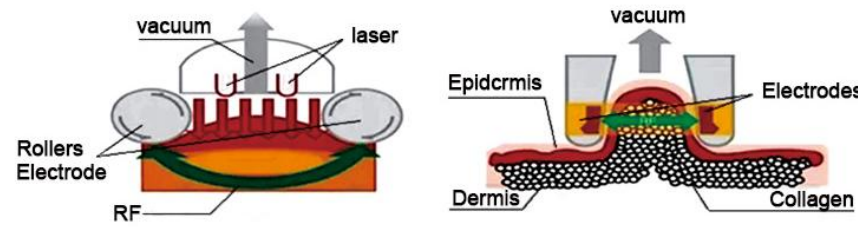

A vacuum elevates the target tissue,bringing it closer to the energy source.
Infrared light directly heats the targeted areas.
Bi-polar radio frequency (RF) energy safety guides heat throughout the targeted area, so that it heals faster without damaging the skin.





(1) Infrared laser reduces skin impedance by heating skin and RF energy penetrates deeply into connective tissue. The synergistic combination of infrared laser and conducted RF energies increases the oxygen intracellular diffusion by heating the skin.

(2) Vacuum plus specially designed rollers manipulate leads RF penetration to be even 5-15mm. At the same time, vacuum and roller mechanical tissue manipulation nips stretches fibrillar connective tissue, effectively breaks down subcutaneous fat as well as extruded capillary vessel, Increase lymphatic drainage, promotes metabolism and reduces or shrinks the size of the actual fat chamber and greatly improved body contouring effect.

(3) The technology that vacuum folds skin makes RF energy penetrate a specific folded skin greatly improve effect and safety, even for the upper eyelid area treatment.



FOUR-IN-ONE

Rotary Negative Pressure RF Machine
+ Inner ball roller Machine






PHYSIO MAGNETO

SUPER TRANSDUCTION PLUS NIRS


- PHYSIO MAGNETO
- Frequency up to 3000hz
- Intensity up to 4 Tesla
- Unique integrated applicator
- Flexible holding arm

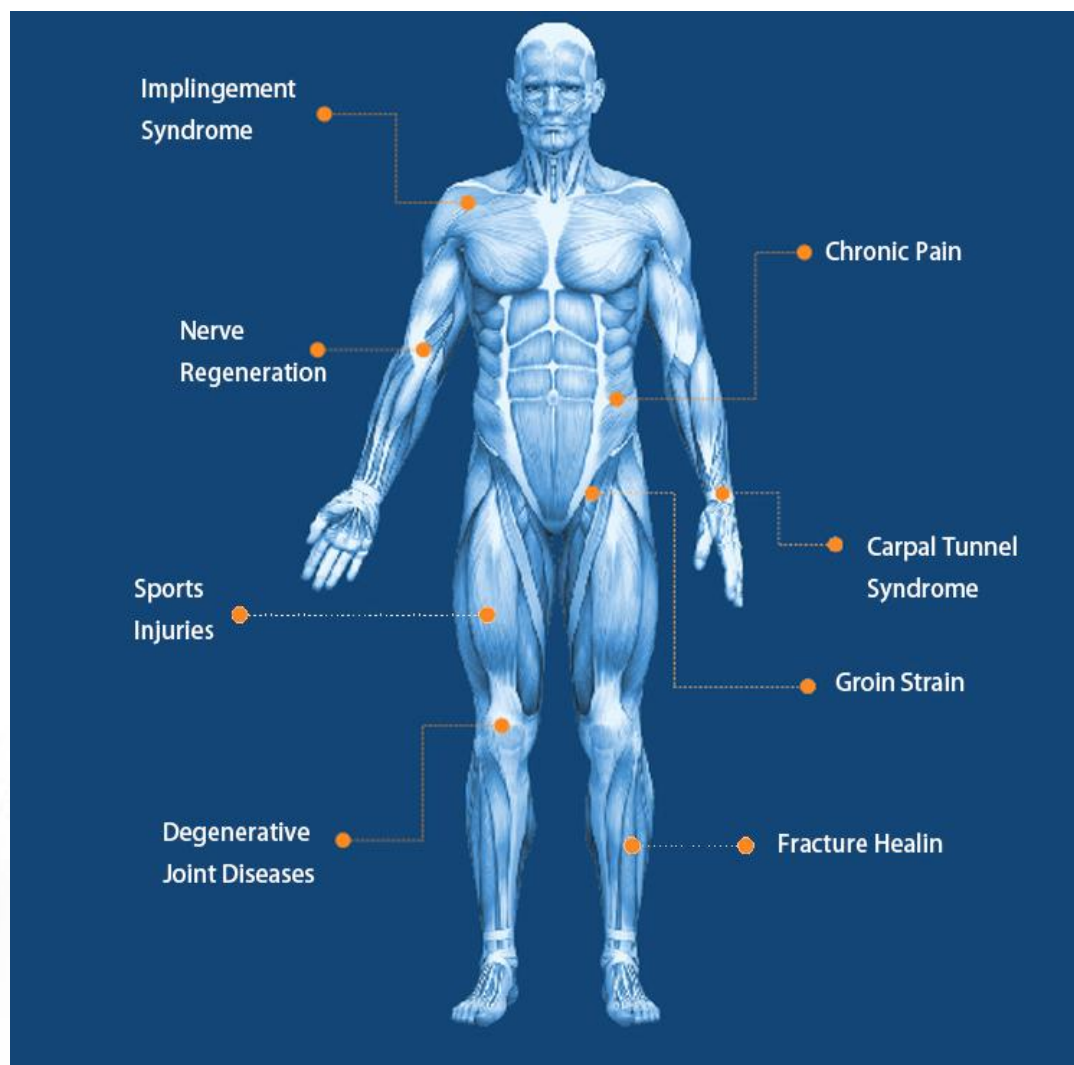

NEAR INFRARED SPECTROSCOPY

NIRS light therapy is a highly effective treatment for all-over wellness, and for treating both acute injuries, chronic pain conditions as well as post-activity recovery.

There is no associated pain or reported side effects, and the technology is very safe with no health risks.

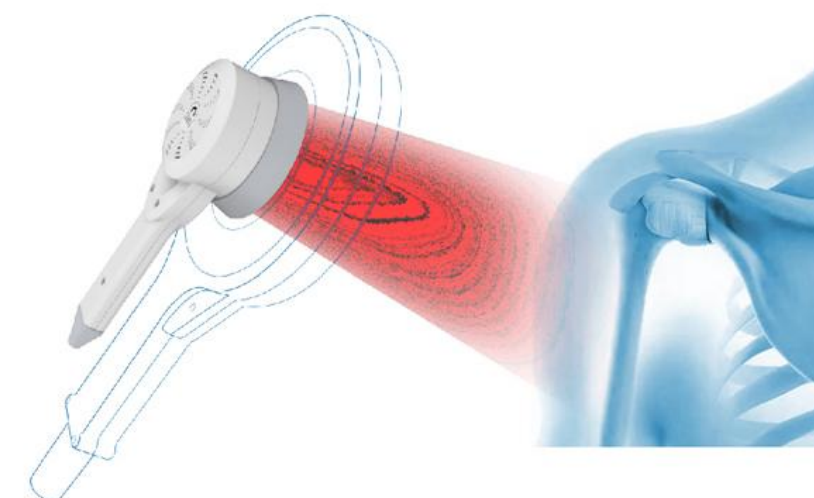

Pulsed Electromagnetic Field Therapy

PMST different with PEMF, it is a ring type coil, cover bigger area and fit to joints part. High speed oscillation for deeper penetration.



Intuitive operation with 10.4 inch touch screen





3 joints holding arm
Easy applicator positioning



UNIQUE RING TYPE COIL TO FIT JOINTS TREATMENT



NEAR INFRARED SPECTRUM FOR DEEP PENETRATION INTO BODY TISSUE

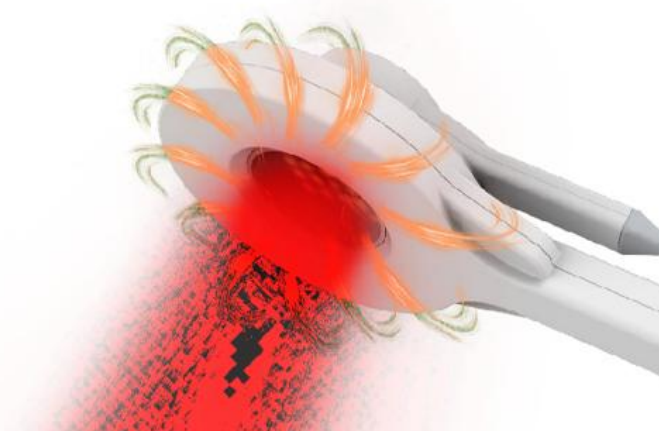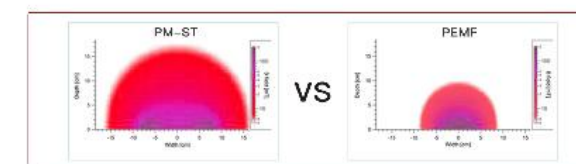



TMS is divided into STMS, PTMS, and RTMS according to the stimulus mode.

STMS: A single-time magnetic field with a non-fixed frequency is used to observe the instantaneous effect, and it is mostly used for conventional electrical physiological examinations.

PTMS: Based on a specific time interval and intensity, 2 stimuli is given to one specific area or two different parts, which are mostly used to study the easilyization and inhibitory effects of the nerves.

RTMS: During a certain area, the magnetic field is changed at a certain frequency. When the stimulus stops, there is still a continuous biological effect. It is a powerful tool for brain function research and clinical treatment.





Clinical Applications



Vitiligo




Psoriasis vulgaris

Palmoplantar psoriasis



palpoe pustular disease

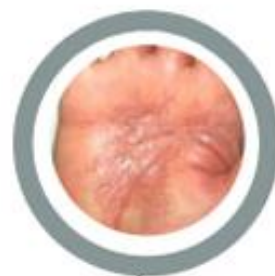

eczema

Main reaction after treatment : (in 2-4h) erythema , some patients without erythema reaction.

Post-treatment reaction : erythema, furfuration, vesicle, pruritus

Treatment interval : 2~3 times/week

Two periods of Vitiligo : aggressive period, stable period

aggressive period



The white spots are increasing, the boundary is blurred and skin is enlarged.

stable period



The white spot stop increasing, the boundary is clear, around the white spot have melanin deepened.

German Lamp, 500 Hours Lifespan


Equips with premium German lamp, which guarantees no energy attenuation. The energy would be stable and accurate to ensure an optimal Clinical result. Best wavelength, high quality lamp and excellent results.

No Recharge, No Consumable

The device is installed with 500 hours illumination lifespan, no need any recharge card Moreover, unlike the 308 excimer laser which needs to replace gas at high cost regularly, Eximal Mini doesn't require any gas. Low running cost, high ROI.

New Replacement under Warranty

During the warranty period, if any malfunction happens due to the material on manufacturing mistake. GSD would provide a new replacement for clients. Free maintenance.

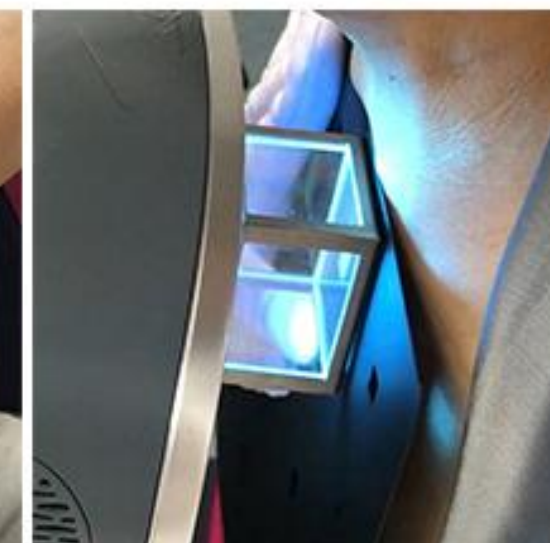



Vitiligo, psoriasis, seborrheic dermatitis, atopic dermatitis, allergic dermatitis and other dermatitis, pityriasis rosea, urticaria, generalized and chronic eczema, and skin pruritus etc.



TECHNICAL SPECIFICATIONS

Lamp Type	XeCl Lamp
Light Wavelength	308nm
Optical Power Density	30mW/cm ² ±20%
Fluence	30-4200mJ/cm ²
Spot Size	Max. 16cm ²
Treatment Time	1s~140s
Erythema Test Dose	3s/6s/9s/12s/15s/18s/21s/24s/27s
Work Mode	Continuous
Dimensions(L×W×H)	27×24×26cm
Power Requirement	100-240V, 47-63Hz

Technical specifications are subject to change without notice.




308nm EXCIMER SYSTEM

The Portable, Highly Effective Solution for Monochromatic UVB Therapy

Psoriasis
Vitiligo
Eczema
Dermatitis



Timecards

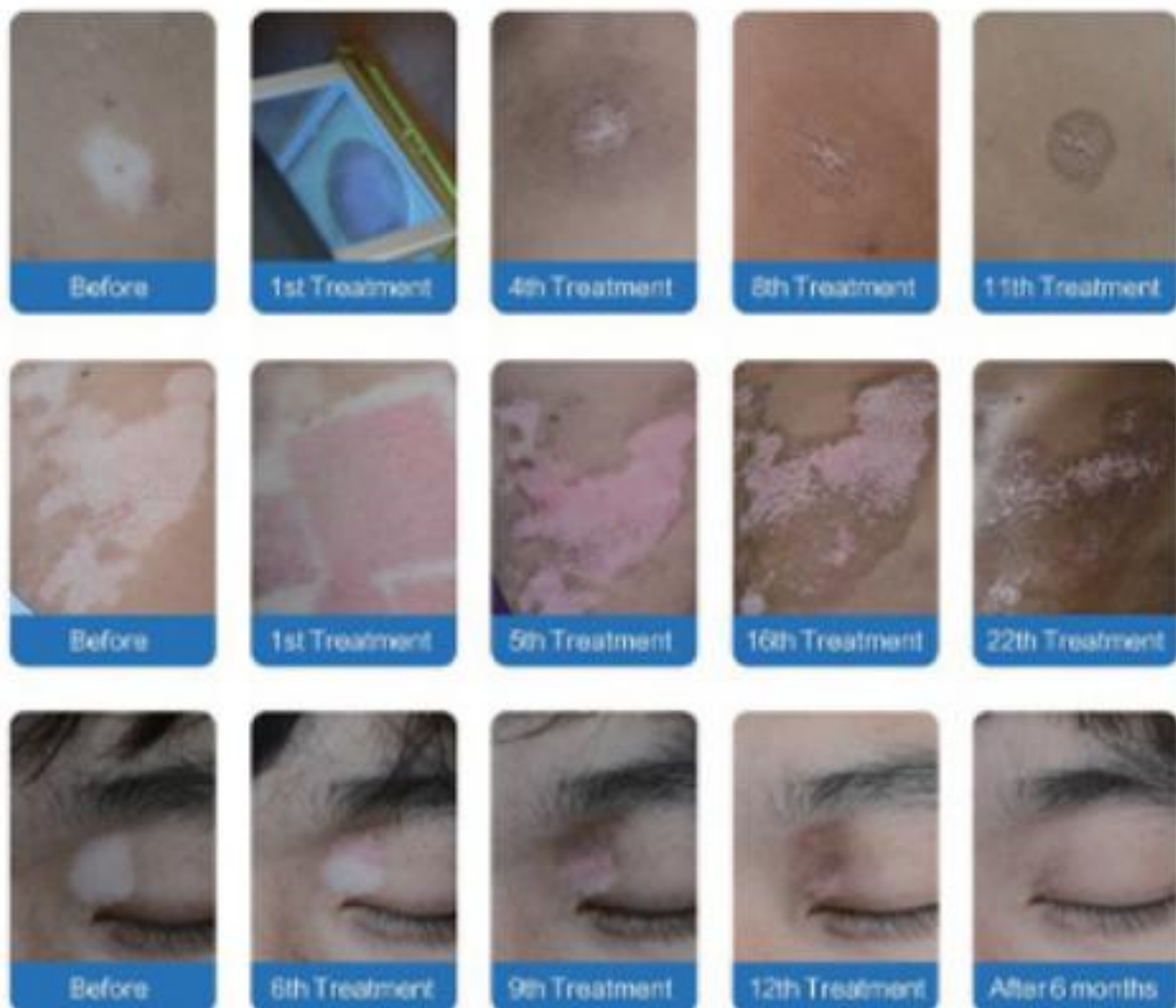



packing list

- Main unit of UV phototherapy instrument × 1
- Light therapy device storage bag × 1
- Power adapter × 1
- Light therapy goggles × 1
- Phototherapy goggles storage bag × 1
- Irradiation window and goggles cleaning cloth X1
- Manual X1
- Product Certificate and Warranty Card X1

The pictures on this page are for reference only, and the final contents are subject to the instructions.



Before



After



Before



After



Before



After



Before



After

Two periods of Vitiligo : aggressive period, stable period



Target Treatment



aggressive period




The white spots are increasing, the boundary is blurred and skin is enlarged.

stable period



The white spot stop increasing, the boundary is clear, around the white spot have melanin deepened.

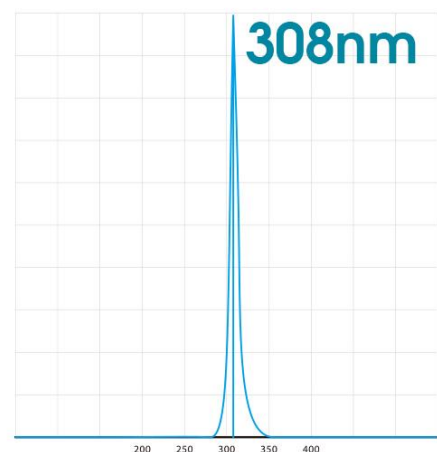

SIGMA 308nm UV light spectrum



Vitiligo



Psoriasis




Dermatitis






SLIMMING SPA CAPSULE

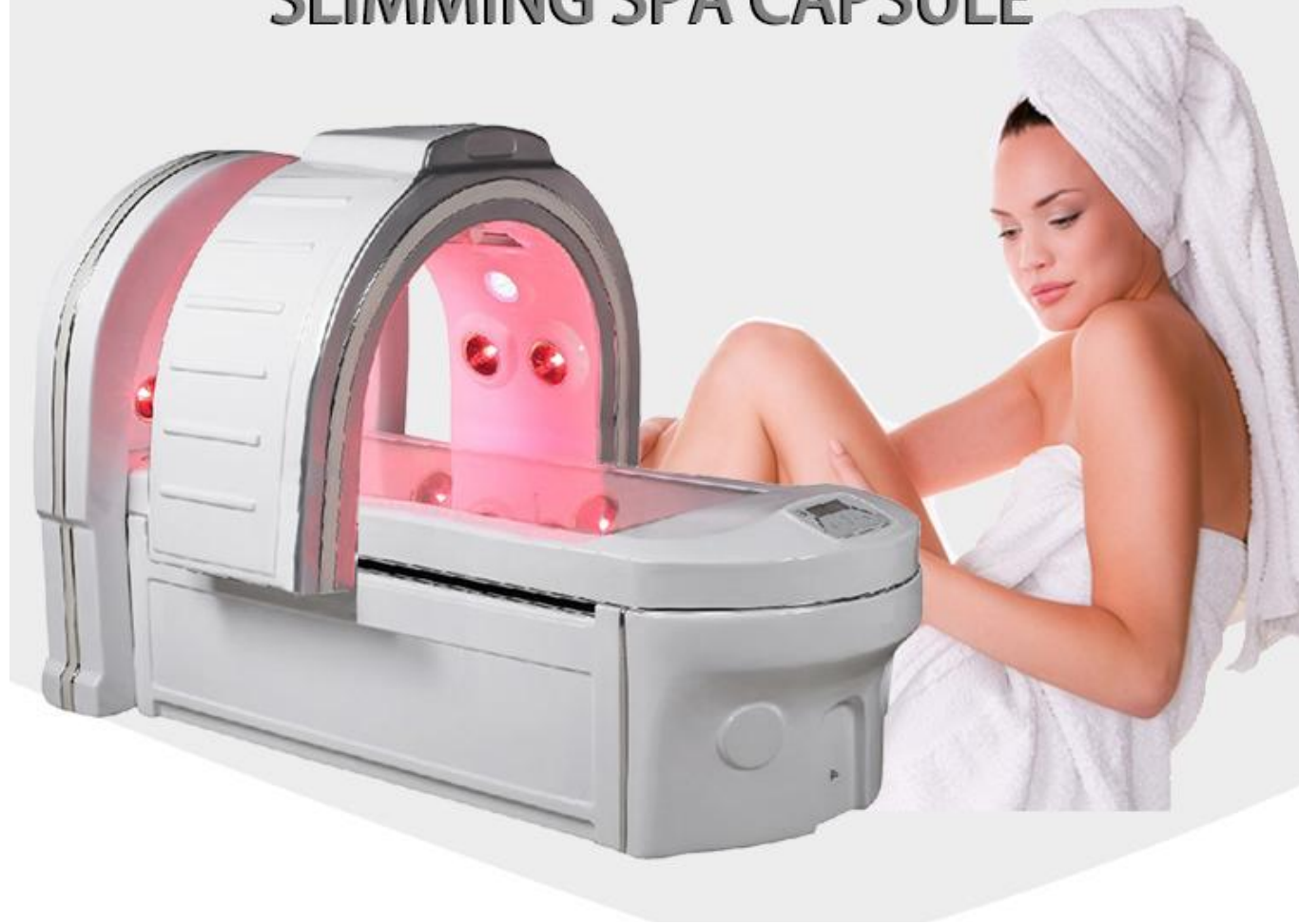

1. Stimulate blood circulation and metabolism.
2. Accommodate intestines and stomach and breathing systems.
3. Eliminate inner waste and prevent disease.
4. Facial, body slimming.
5. Good for neurasthenic, insomnia.
6. Avoid fatigue, recover happy spirit.
7. Adjust nerve function.
8. Lifting the immunity of urinary system, incretion system.
9. Good for cure heart disease, hypertension, diabetes, skin disease neuralgia and rheumatism.
10. Good for valetudinarianism people's rejuvenation.

Use our product usually; it's good for the accommodation of skin, whelk, fatness, limply, cancer-prevention, hypertension myocardial infarction, liver disease and uric acid.

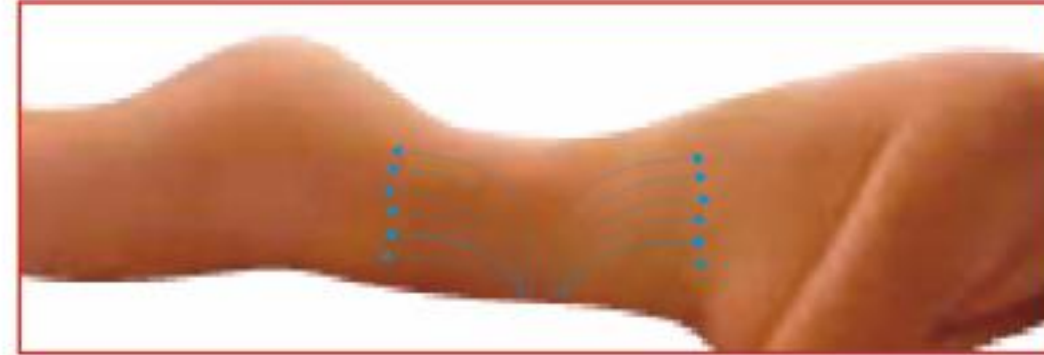



- Eliminate fatigue
- Strengthen function
- Relaxing tendons and collaterals
- lymphatic drainage
- to promote metabolism





980nm 5+1 diode laser machine

Vascular Removal • Physiotherapy • Nail Fungus Removal • Skin Rejuvenation • Eczema herpes



parameter | 980nm 5+1 diode laser machine



Machine 1



Vascular removal plug 2



Ring for adjusting the focal length 5



Nails fungus removal plug 1



Physiotherapy plug 1



Eczema herpes plug 1



Skin Rejuvenation plug 1



Ice compress hammer 1



Power Line 1



Foot switch 1



Laser safety glasses 1



Eye mask 1



Acrylic bracket 1



Manual 1



Flight case 1



Nails fungus removal

Onychomycosis is a fungal infectious disease that occurs on the deck, nail bed or surrounding tissues. It is mainly caused by dermatophytes. Onychomycosis is highly contagious and is the most stubborn and refractory of dermatophytosis. For its treatment, there are currently medical treatments, surgical treatments and laser treatments. Laser treatment is a new type of treatment. It uses the principle of laser to irradiate the disease with a laser to kill the fungus without destroying the normal tissue. It is suitable for all kinds of nails.

II 980NM DIODE LASER SPIDER VEIN REMOVAL MACHINE

Treatment theory


The instrument uses a 980nm wavelength laser to transmit laser energy through the fiber to the blood vessel site. According to the principle of selective light and heat, hemoglobin absorbs the laser energy and converts it into small molecular clusters to be absorbed by the tissue. The occupied space is covered with newly-grown skin tissue, so as to achieve the purpose of eliminating vascular.



**Work immediately! Work after being cured!
Targeting non-invasively and leaving no scars!**



Onychomycosis is a fungal infectious disease that occurs on the deck, nail bed or surrounding tissues. It is mainly caused by dermatophytes. Onychomycosis is highly contagious and is the most stubborn and refractory of dermatophytosis. For its treatment, there are currently medical treatments, surgical treatments and laser treatments. Laser treatment is a new type of treatment. It uses the principle of laser to irradiate the disease with a laser to kill the fungus without destroying the normal tissue. It is suitable for all kinds of nails.

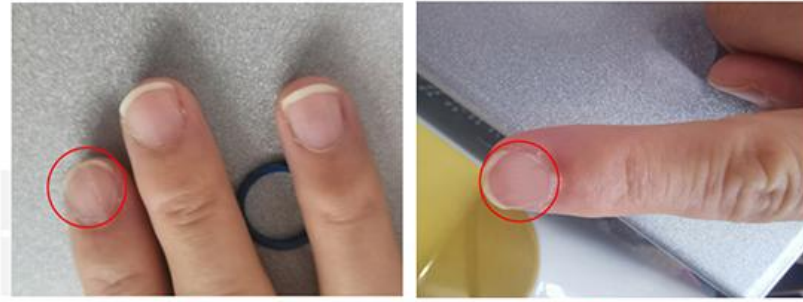

Before After



The instrument focuses on the lens to produce acupuncture and moxibustion on the human body. According to medical principles such as anatomy, neurophysiology and traditional Chinese acupuncture, infrared light is applied to the nerve roots, nerve trunks, ganglia, acupoints and local pain points of the human body. To adjust the stability of the human body's environment, so that each system tends to a normal physiological balance, thereby achieving the purpose of treating diseases and preventing diseases.



Laser treatment of eczema



Laser combined with drug treatment is not only suitable for herpes zoster, but also for skin ulcer, urticaria, psoriasis, eczema, etc., with satisfactory effect, short course of treatment, high cure rate and low recurrence rate.



Case1 Before treatment After twice treatments



Case2 Athlete's foot itching, significantly improved the next day after laser irradiation.

Ice compress hammer



Reduce the temperature of local tissues in the body.
Shrink blood vessels.
Reduce swelling and pain to the greatest extent.
Reduce the sensitivity of tissues to pain.





Laser treatment should be done immediately ice compress

Skin Rejuvenation Anti-inflammation



980 nm laser rejuvenation is a non-exfoliating stimulation therapy. It improves the skin quality from the basal layer. It provides non-interventional treatment, and is suitable for different skin states. It penetrates the skin about 5 mm thick through a specific wavelength, and reaches the dermis directly, which acting directly on collagen cells and fibroblasts in the dermis. Protein of skin can be regenerated under the stimulation of weak laser. It can really achieve the function of skin care. It will not cause any damage to the skin.

980 nm laser irradiation can also dilate capillaries, enhance permeability and promote the absorption of inflammatory exudates. It can improve the phagocytosis function of leukocytes, so it can affect the activity of enzymes and regulate the immune function of the body, Then finally achieve the purpose of anti-inflammation, anti-swelling and accelerate the process of tissue repair.

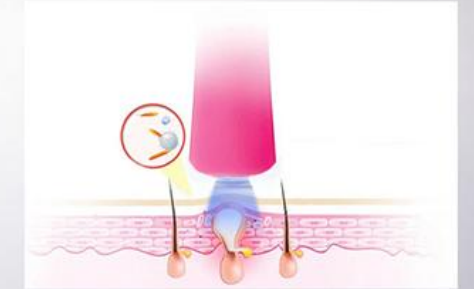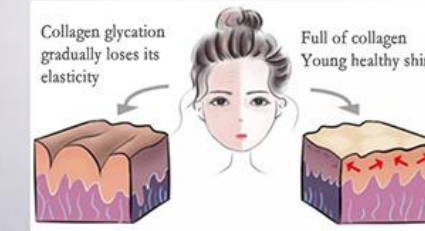

Laser treatment of dysmenorrhea

The 980nm semiconductor laser has strong penetrating power to tissues, has analgesic and autonomic nerve regulating functions, can dilate blood vessels, improve blood circulation of the uterus, relieve uterine smooth muscle spasm, and achieve analgesic effect.



The picture to show acupuncture points: Sanyinjiao, Xuehai, Ligou, Zhongji, Qihai.





PHYSIOTHERAPY

- ▶ Red Blood — 980nm、1470nm、RF
- ▶ Grey nails — 980nm、1064nm、980nm+810nm
980nm+1064nm、
- ▶ Pain Physical Therapy — 980nm、810nm、1064nm、
980+810nm、980nm+1064nm
980nm+810nm+915nm+1064nm
- ▶ Skin rejuvenation and anti-inflammatory — 980nm、980nm+1470nm
- ▶ Sorrow away, gynecology — 980nm、1470nm、
980nm+1470nm

SURGERY

仅出口！仅供专业人士！

- ▶ Lipolysis — 980nm、810nm、1060nm、1470nm
980nm+810nm、980nm+1470nm
- ▶ Varicose Veins (EVLV) — 980nm、810nm、1060nm、1470nm
980nm+810nm、980nm+1470nm
- ▶ Hemorrhoids and other cutting class — 980nm、1470nm
980nm+1470nm

DENTISTRY

- ▶ Tooth whitening
Periodontitis, etc. Inflammation
Root canal treatment, cutting, etc. — 980nm, 810nm,
1064nm, 450nm



PETS

980nm, 980+810nm, 980+810+915+1064

- ▶ Pets Physiotherapy: Pain Relief & Trauma Treatment. It can relieve pain caused by various injuries, fractures, etc. in cats/dogs and other pets, reduce inflammation and swelling, promote healing and delay aging. Commonly used to treat osteoarthritis, degenerative joint diseases, joint dysplasia, skin diseases, acute and chronic ear infections, periodontal disease, post-operative healing and acute trauma.
- ▶ Pet surgery category: Surgery: surgical incision, surgical dissection, evaporation, vascular sealing/cutting, vascular sealing (ligation), thermotherapy, laser ablation, glaucoma, eyelash impingement, periodontal disease, debridement, cystitis, anal glands, claw extraction, oral melanoma, extension of ear canal, chronic ulcers - wound healing, eyelid tumors, tumors, etc.



